MOBİL YOĞUN BAKIM KABİNİ

MOBILE INTENSIVE CARE CABIN

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Due to the fact that the COVID-19 virus that occurs in the year 2020 creates a pandemic, it is a complicated device that our company has produced.

It is intended to be used if hospitals are inadequate.

Through this system, it is aimed to raise the standard of life of the person who has the disease and to protect the health of the medical personnel who take care of the patient at the maximum level.



It seems that humanity could be at risk of this kind of outbreak and transmission at any time. Even if COVID 19 is on the agenda today, different viruses will need different treatments in the future.

For these reasons, the devices that provide patient care and treatment are not fixed in our device.

It is aimed to change the devices to be used according to the type of disease and the treatment needed.

With this device, gyms,sheds, warehouses, factories and fair areas can be transformed into hospitals.

If COVID 19 is to become more common,faster infected, such a device is always needed, no matter how much more hospital numbers and intensive care beds.

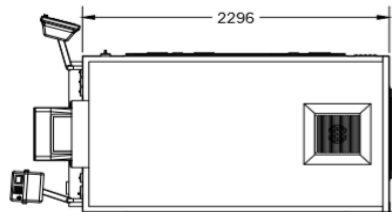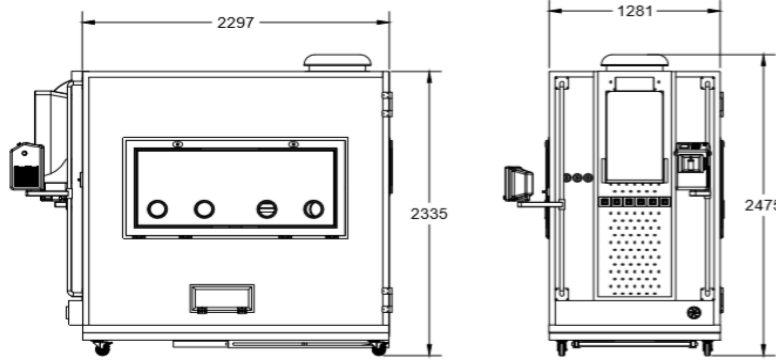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SRT
MARCHIA
www.srtvolmak.com

1- VENTILATOR HOSE AND CABLE TRANSITION : It is a medical device that is not attached to the cabin. Essential for patients who are intubated. It has a 100mm diameter sealed window. It consists of 4 parts. Sealed gasket of cable switch is an apparatus that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on Page-1, the detail on page-1)

2- OXYGEN CONCENTRATOR TRANSITION : The oxygen concentrator is a transition window to connect its device. It consists of 4 parts. Sealed gasket of cable switch is a window that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on Page-1, the detail on page-1)

3- FEEDING PUMP TRANSITION : The feeding pump is a transition window to connect its device. It consists of 4 parts. Sealed gasket of cable switch is a window that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on page-1, the detail on page-1)

4- PERFUSION PUMP TRANSITION : The perfusion pump is a transition window to connect its device. It consists of 4 parts. Sealed gasket of cable switch is a window that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on page-1, the detail on page-1)

5- INFUSION PUMP TRANSITION : The infusion pump is a transition window to connect its device. It consists of 4 parts. Sealed gasket of cable switch is a window that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on page-1, the detail on page-1)

6- CABIN DOOR WINDOW : It is fixed on the door, sealed,intended to provide psychological relief according to the direction of view of the patient. (The image on Page-2)

7- TELEVISION : 37 " it is intended to enable the conscious patient to pass the time. (The image on Page-2)

8- CABIN RAMP : The wheeled one allows the patient's bed to enter the cabin. It is manufactured from Crni. In cases where the ramp is not needed, it is sliding into the underside of the cabin. (The image on Page-2)

9- PATIENT BED CONTROL : It is mounted outside the cabin so that the paramedic can get the patient to the position he or she wants. (The image on Page-3)

10 - MEDICAL WASTE WINDOW : It is the window where the materials used for the patient and the waste that the patient must dispose of are discarded. Sealed plastic bag suitable for this window is used by passing it on the outside. (The image on Page-2)

11- FOOD SERVICE WINDOW : It is a sealed window for conscious patients to be given what the patient can eat. (The image on Page-2)

12 - PATIENT INTERVENTION WINDOW : The window is 130mm in diameter. Total 6 pieces. Used by wearing patient intervention gloves, by cutting off physical contact with the patient, it protects the person who intervenes with the patient. (The image on Page-2)

13 - CABIN RIGHT WINDOW : It is on the right side of the cabin, made of PVC sealed window.In order to provide direct physical contact to the patient, it can be opened in a complete downward direction.In this case, the health personnel should take the necessary precautions because they will be unprotected. (The image on Page-3)

14- HEAD OF PATIENT MONITOR CABLO INPUT : Located on the right side of the cabin is a window opened close to the head of the patient. It consists of 4 parts. Sealed gasket of cable switch is a window that allows the patient to receive treatment by opening holes according to the desired diameter and quantity. (The image on page-3, the detail on page 1)

16 - SHEET OF CABIN CEILING DECOR : It is a piece made from Crni in order to hide the ventilation, lighting, UVC lamp and air conditioning in the cabin and to provide a good look to the patient. (The image on Page 3)

17 - RAMP OUTSIDE TRACK : It is the part that allows the ramp to settle under the cabin. (The image on Page 3)

18 - WHEEL OF CABIN : Located under the cabin that is 4 wheels with brakes. Allows the cabin to move in the field. (The image on Page 3)

19 - POTTY SLIDER WINDOW : It is a sealed window used to take the potty from under the potty beds. (The image on Page 4)

20 - CABIN VENTILATION : At the top of the cabin is a window with a 100mm diameter filter. Provides air for the in-cabin environment. (The image on Page 4)

21 - TRANSITION PART BETWEEN AIR CONDITIONER CABIN : It is a part of the air conditioning that allows the hot or cold air to pass between the ceiling of the cabin and the decor sheet. (The image on Page 4)

22 - AIR CONDITIONING CONNECTION RODS : It is the apparatus that connects the air conditioning to the number 25 part located at the back of the cabin. (The image on Page 4)

23 - ELECTRICAL OUTLETS OF CABIN : They are sockets installed in part 25 that provide the electrical requirement of medical devices. (The image on Page 4)

24 - UNIT OF VIRUS BURNING : It is a device used to destroy viruses and microbes that accumulate or are found in the cabin. It contains a 33W UVC lamp. Optionally, an incinerator capable of generating over 100 degrees of heat is installed and allows the destruction of viruses and germs. (The image on Page 4)





25 - DEVICE CONNECTION SHEET : It is an apparatus manufactured from Crni for mounting some medical devices to the cabin which cannot be connected to the arms and for laying the cable system.(The image on Page 4)

26 - DEVICE CONNECTION RODS : It is a 2-piece mounting arm that allows the medical devices needed for the patient to be installed in the Cabin. (The image on Page 4)

27- FAN OF BURNING UNIT : It is a device that allows the air inside the cabin to be released to the outside environment by passing through the virus burning unit and at the same time creating negative pressure inside the cabin. (The image on Page 4)

28 - RAMP SLIDER HANDLE : It is a part of the ramp that provides mobility. (The image on Page 3)

29 - CABIN LEFT INTERVENTION WINDOW : On the left side of the cabin is a sealed window made of PVC. In order to provide direct physical contact to the patient, it can be opened in a complete downward direction.In this case, the health personnel should take the necessary precautions since they will be unprotected. (The image on Page 2)

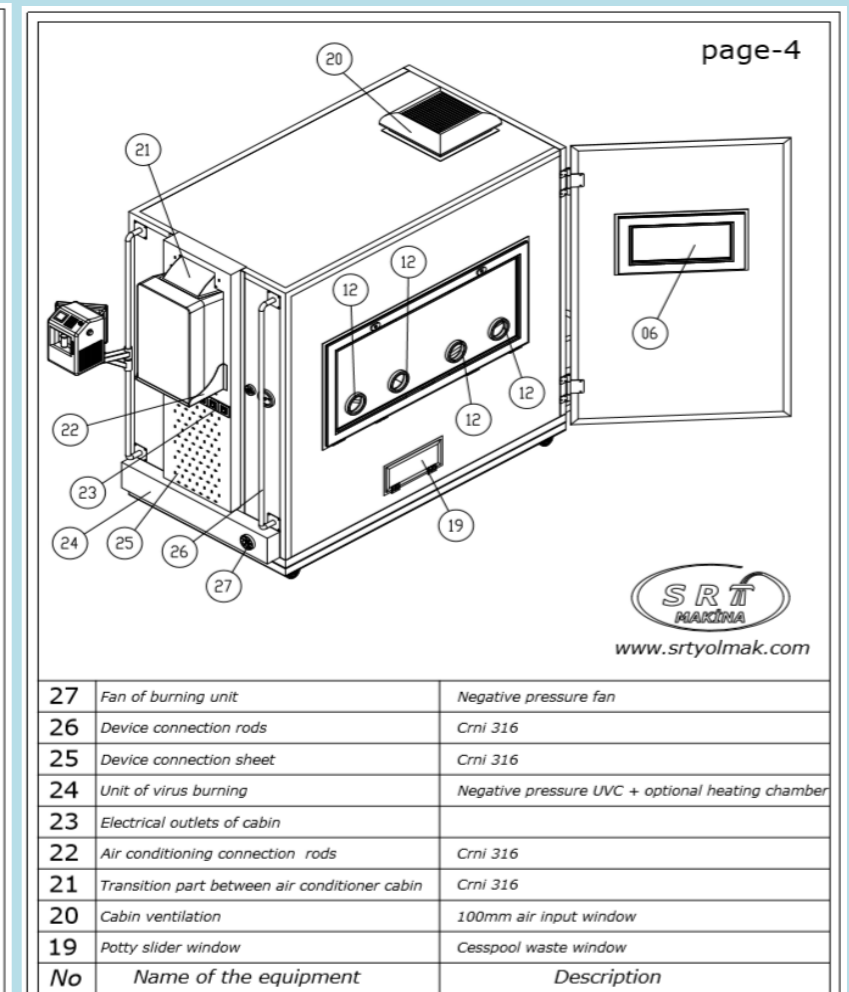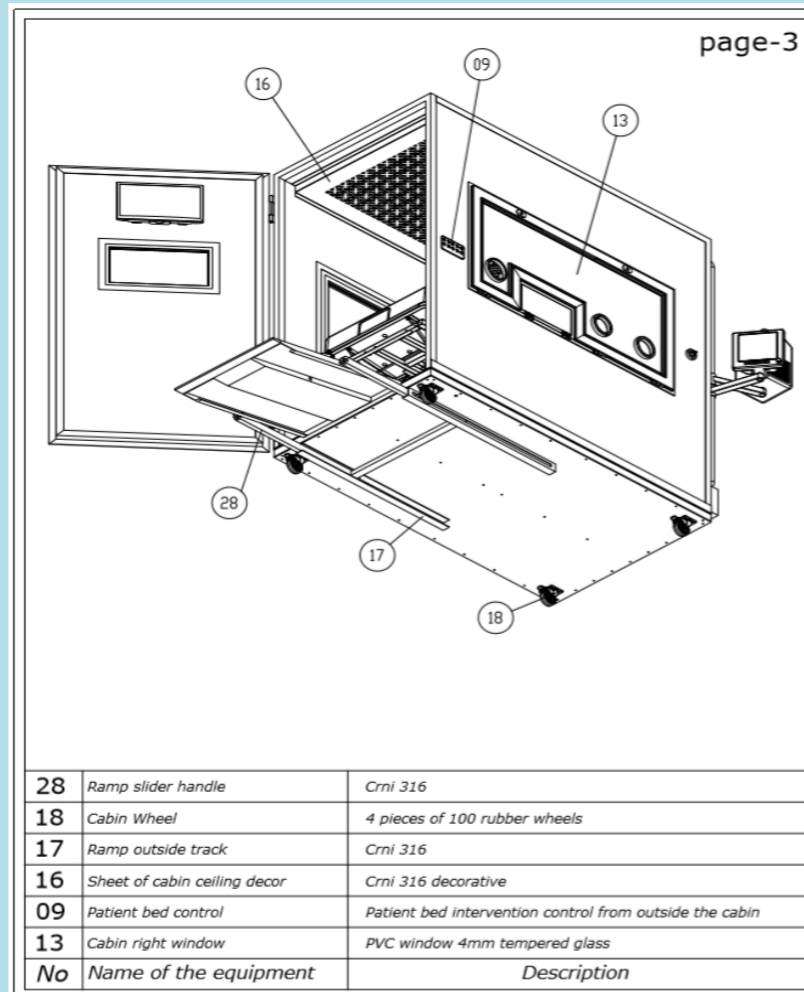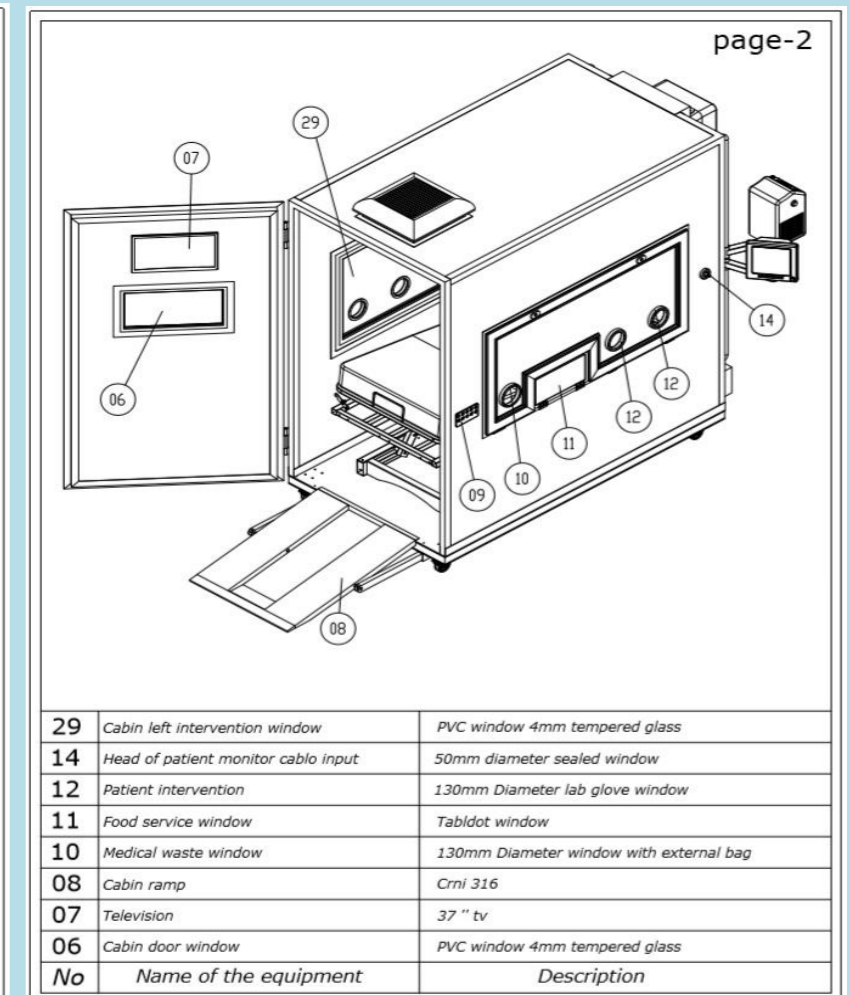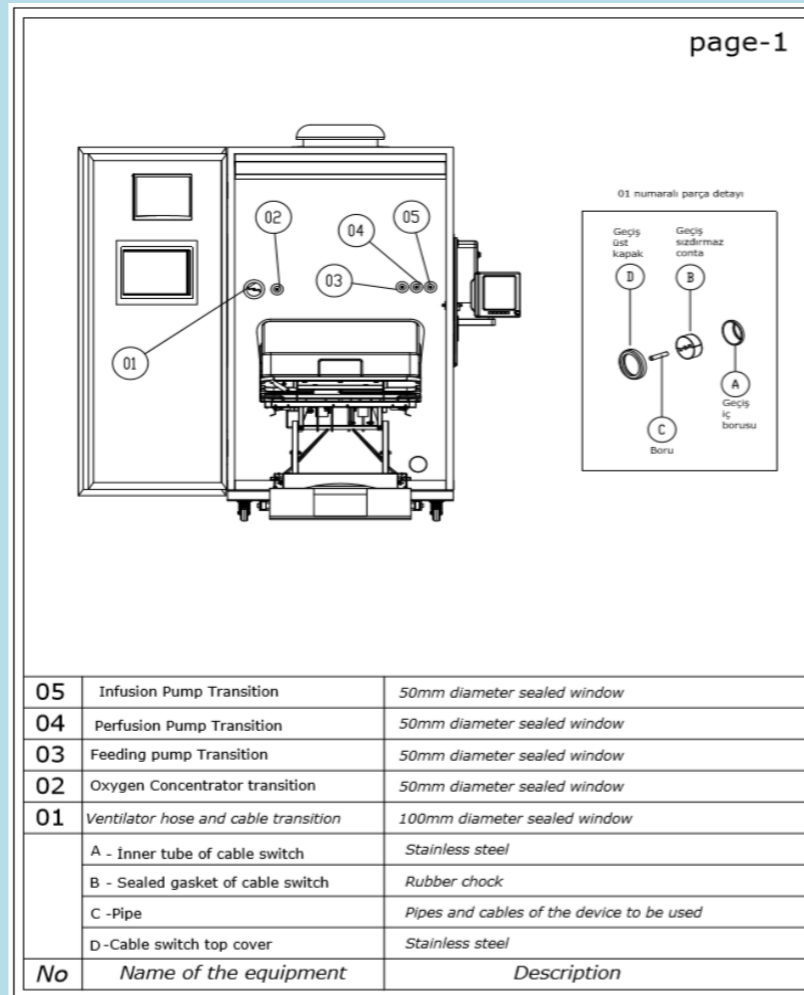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MOBILE TOWER SYSTEMS

Our tower systems will serve in many fields as well Especially;

- * Providing continuous communication in AFAT situations
- * Transmitter Towers
- * Lighting Towers
- * Military areas


* Can be used in many areas that the private sector needs.

The height of the tower is between 6000mm and 33400mm.

The foot openings cover large areas against tipping.

Furthermore, after the tower is fixed, it is connected with safety ropes in three directions.

There is a living space inside the control cabinet to meet the needs of a staff member.

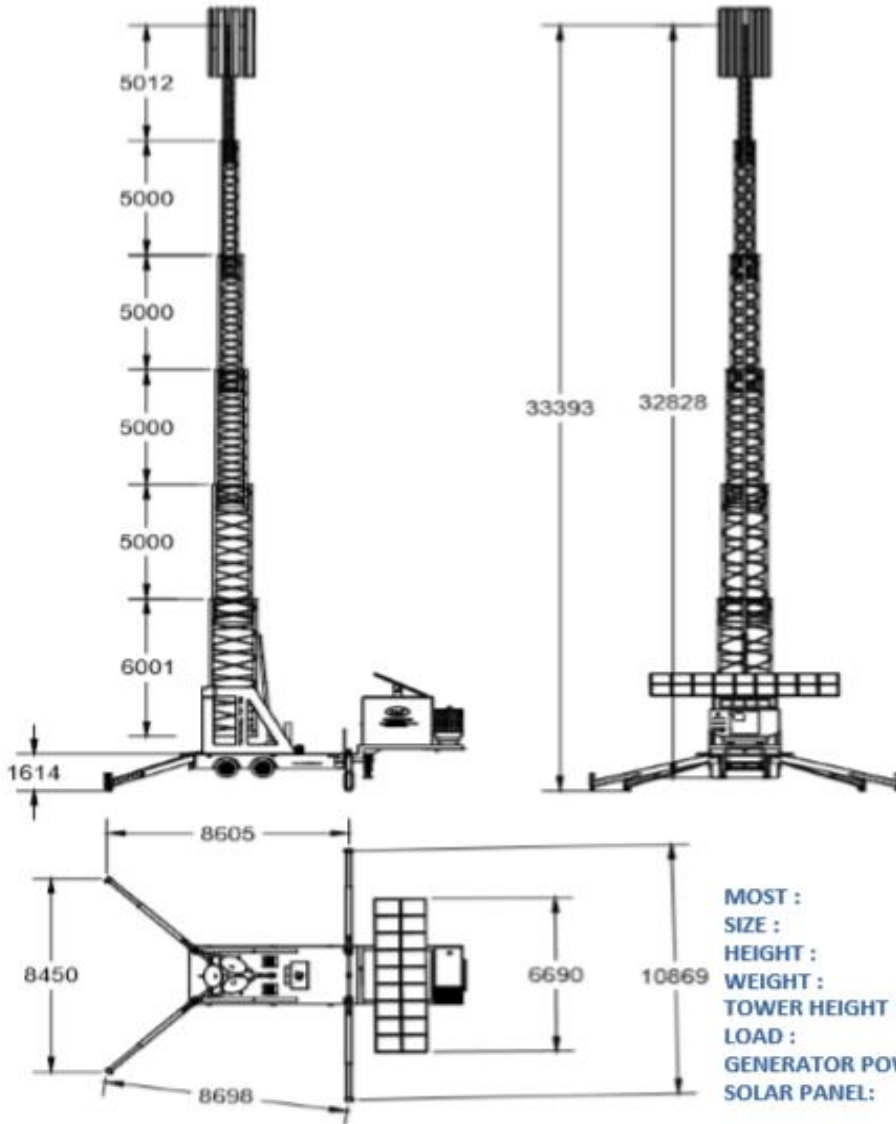DIMENSIONS

TRAVEL	SETUP
2550 mm	10900 mm
4000 mm	11300 mm
	33400 mm
	10250 kg
	6000 - 31000 mm
	500 kg
	33 - 70 kVA
	UP TO TOWER POWER NEED

www.srtyolmak.com

SRT MADEN MAKİNA LTD. ŞTİ. Ortakonak Mah. 14332 sokak no: 49 Karatay - KONYA
Tel : 0332 353 29 98 - 0 542 521 90 30